HyStool![®]

100% Water Soluble Stool Sample Collection Bags

Flushable, Hygienic, Biodegradable
Environmentally Friendly

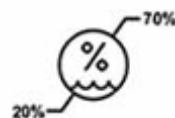
Instructions for use

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Instructions d' utilisation

HyStool Ltd, Aberlady, Scotland, Great Britain

Sungro Europe B.V, Netherlands

www.hystool.com

1. Go for a Pee before using.

HyStool bags are 100% water-soluble and can dissolve if they touch water.

2. HyStool bags stick to the toilet seat. The seat must be dry.

If the seat is damp, the stickers may not stick properly. If necessary, dry the seat before using the bag.

3. The HyStool bag comes folded inside a pack.

Take it out and unfold it completely before attaching it to the toilet seat.

4. Peel off backing from two stickers on the wider edge of bag and attach them to the rear of the toilet seat.

The folded edges of the bag are positioned to the sides of the toilet seat.

5. Peel backing paper from remaining two stickers stretch bag forward and attach to seat as shown.

Leave a gap at the front so any accidental discharge of urine misses the bag.

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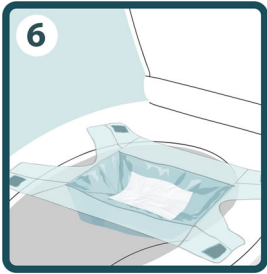
6. Place some toilet paper in bottom of the bag.

Toilet paper helps prevent a wet stool from touching the bag. The bag may dissolve if it gets wet.

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7. Use toilet as normal.

After your bowel movement collect the stool sample from the HyStool bag as soon as possible.

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8. collect a stool sample from the HyStool bag using a collection device.

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9. Peel the stickers from the toilet seat and let the bag and contents to fall into toilet water.

Simply Flush Away. HyStool bags are 100% water soluble.

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10. If the bag doesn't flush the first time, flush again. Any residue in the bowl can be brushed away.

If sticker marks are left on the seat, wipe them off with a damp cloth.

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